

CERTIFICATE OF AUTHENTICITY

CARE INSTRUCTIONS FOR:

LAB:

PROVIDED BY:

Boston[®]
Materials



Your custom contact lenses are coated with **Tangible[®] Hydra-PEG[®]** technology to enhance their wettability. **Tangible Hydra-PEG** has been shown to increase surface water retention and lubricity, and minimize deposits on various lens materials.

To best care for your **Tangible Hydra-PEG** coated lenses, it is very important to follow the Lens Care and Handling Instructions provided on the back.

CONGRATULATIONS ON YOUR NEW LENSES!

LENS CARE AND HANDLING INSTRUCTIONS



CAREFUL! LENSES ARE SLIPPERY

Plug your drain to avoid losing your lenses



DON'T USE TAP WATER

Only use buffered, preservative-free sterile saline solution to rinse your lenses



DON'T USE ENZYMATIC, ABRASIVE, OR ALCOHOL- BASED CLEANERS

They may damage the Tangible Hydra-PEG coating

**USE ONLY RECOMMENDED
SOLUTIONS AND CLEANERS**

e.g., Boston Simplus® Multi-Action Solution



Boston
Materials

 **tangible**
HYDRA-PEG®

Visit fit-boston.eu for important safety information.

Tangible and Hydra-PEG are trademarks of Tangible Science, LLC used under license.
Refer to your package insert for additional instructions. If unsure, consult your Eye Care Professional.

© 2020 Bausch Health Companies Inc. or its affiliates.